



This device was designed to allow aides to lift persons from a sitting without putting stress on the persons arms and shoulders. the lift is placed on the seat prior to sitting down. because the lift is not bulky, it may be sat on without irritation. the webbing may be washed if soiled. Cotton webbing is available at most fabric stores.

Length of webbing may be increased for larger person.

Material required:

(4) Yards 7/8" Cotton webbing
Thread

Cut webbing:

- (1) 72" strip
- (8) 9" strips

Sew webbing as shown on sheet 2

1. Assemble straps in crosshatch pattern
2. Zigzag stitch along end of all straps to prevent fraying
3. Straight stitch all seams about 1/2" apart for reinforcement
4. Overlap 72" strap about 3" and seam using zigzag and straight stitches

Project Name: Lifting Web

Drawing Name: Plans & Instructions

Submitted By: J. Weisman

Scale: Not to Scale

Date: 11.7.13

Sheet #: 1/1

Rev: A